

The first and only FDA authorized medical device that helps reduce the symptoms of fibromyalgia

Quell | Fibromyalgia



What is Quell Fibromyalgia?

An easy to use, wearable neuromodulation device you can prescribe to your patients with fibromyalgia.

How does it work?

Quell provides symptom relief by comfortably stimulating sensory nerves through a device worn around the calf. The nerves carry neural pulses to the brain that trigger a natural response believed to reduce the body's hypersensitivity to pain and other symptoms associated with fibromyalgia.*



How is Quell Fibromyalgia different than traditional TENS?

Quell's technology is a closed loop system, consistently monitoring multiple physiological parameters and adjusting stimulation to ensure a consistent treatment dose. Quell is true wearable neuromodulation that can be worn for long periods of time and is approved for use during sleep.

Side effect profile

Since this is a non-invasive, wearable device, there are no significant side effects. Some patients may experience minor skin irritation.

Contraindications

The device is contraindicated for use by patients who have a cardiac pacemaker, implanted defibrillator, other implanted electronic device, or implanted metal near the device.

Recommendations for use

New users should wear Quell for a minimum of 3 treatment sessions per day for the first 30 days. Thereafter, they may use as often as needed for symptom relief. Quell may also be used overnight, during sleep.

Device placement

Quell is designed to be worn on the upper calf, 1-2 inches below the knee. It can be worn on either leg.

Device cost

Quell Fibromyalgia is not covered by insurance and is only available on a cash pay basis. The starter kit is \$75 and includes everything a patient will need to evaluate the device (Device, strap, charging cable, 30-day supply of electrodes). If the patient would like to continue after the evaluation period, they just need to purchase electrode refills, which are sold in a 3 month pack for \$199.

How to prescribe?

Simply complete and submit the attached prescription form to HealthWarehouse, our exclusive online pharmacy. HealthWarehouse will contact your patient to collect required information and payment details.

Device shipping + setup

The patient will receive the device in the mail and be guided through an easy, step-by-step setup via our mobile app. If they have any questions during the process, they can speak with a member of our Boston-based Customer Care team at 800-204-6577.

If there are other patients in your practice who could benefit from Quell Fibromyalgia, contact us at info@quellfibromyalgia.com for additional resources. Display devices, patient brochures and prescription pads are available upon request. For more information on Quell Fibromyalgia, visit: www.QuellFibromyalgia.com.

**The specific biological mechanisms by which Quell produces all its clinical benefits are uncertain. Additional support from unpublished data in fibromyalgia and chronic low back pain RCTs.*



STARTER KIT

Includes:

- 1 Quell Fibromyalgia Device
- 1 Quell Standard Size Band
- 1 Month Supply of Standard Electrodes (2 Electrodes)
- 1 Charger
- 1 Charger Cable

PRESCRIBER FAX FORM

1. PATIENT INFORMATION

Patient's Full Name _____ Patient's Birthdate (MM/DD/YY) _____

Address _____

City _____ State _____ Zip _____

Phone Number _____ Email _____

ADDITIONAL COMMENTS OR INSTRUCTIONS:

2. MEDICATION INFORMATION

QUELL FIBROMYALGIA STARTER KIT

Prescription Device _____

DIRECTIONS REQUIRED

As Needed Other Directions _____

3. PRESCRIBER INFORMATION

Prescriber's Name _____

Address _____

City _____ State _____ Zip _____ DEA, NPI, or License # _____

Phone Number _____ Fax Number _____

Prescriber Email* _____

*to be used for questions about this prescription or manufacturer updates concerning the device

ADDITIONAL COMMENTS OR INSTRUCTIONS:

Prescriber's Signature (REQUIRED) _____

ALL FIELDS ARE REQUIRED FOR A VALID PRESCRIPTION.
FAX FORM TO (859) 525-0198

Notice of Confidentiality: The information in the facsimile transmission contains confidential information belonging to the sender that is legally privileged and is intended only for the individual designated above. Others are hereby notified that disclosure, copying, distribution, or taking action based on the content of this information is strictly prohibited. If you have the facsimile in error, please notify the sender immediately at the number provided below. Any facsimile erroneously transmitted to you should be returned to the sender by US mail, or if authorized by the sender, destroyed.

TRY.HEALTHWAREHOUSE.COM/QUELL

HealthWarehouse.com

7107 Industrial Road, Florence, KY 41071

Phone: (855) 446-4375 | Fax: (859) 525-0198